

SALIVA PACKAGING INSTRUCTIONS

- Once collected, the subject should place the collection tube back in the blister wrap. The blister wrap may then be labeled with subject information or labels can be placed directly on the tube prior to putting back in the packaging.
- The sample should then be labelled with the patient's:
 - a. First name
 - b. Last name
 - c. Date of birth
- After labelling the sample, the tube and blister pack should be placed in an individual biohazard bag.
- Once all samples are in biohazard bags, they should then be placed into a larger bag for shipment.
- All samples should be loaded into the Styrofoam shipping containers that have been provided.
- Store and transport collected saliva at room temperature. DO NOT store or transport on ice/dry ice.**

